SIGVARIS COMPRESSION STOCKINGS

Product Catalog

Find us on social media
SIGVARIS GROUP INDONESIA

Selection of compression class

CCL 1

18-21 mmHg

- Heavy, tired legs with a tendency to swelling in the legs
- Prevention of thrombosis and embolism in immobile patients
- Prevention of Economy class syndrome (deep vein thrombosis)
- Superficial varicose veins without leg oedema
- Superficial varicose veins during pregnancy

CCL 2

23-32 mmHg

- Varicose veins with mild leg oedema
- After varicose vein treatment (surgical procedures, sclerotherapy, endovenous treatments) to maintain treatment success
- Deep vein thrombosis
- Post-thrombotic syndrome
- Aseptic superficial thrombophlebitis
- After the healing of venous ulcers in patients with chronic venous insufficiency
- Severe varicose veins during pregnancy

CCL 3

34-46 mmHg

- Active venous leg ulcers
- Recurrent venous leg ulcers
- Forms of advanced CVI such as lipodermatosclerosis
- Reversible lymphoedema, lipoedema
- Post-traumatic syndrome
- Angiodysplasia

CCL 4

min 46 mmHg

- Irreversible lymphoedema
- Severe post-thrombotic syndrome

Stocking with the indications for class 3 and 4 should only be prescribed by medical specialists

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Indikasi :

- Varises
- Pencegahan & Treatment CVI (Chronic Venous Insufficiency)
- Pencegahan & Treatment DVT (Deep Vein Thrombosis)
- Post Sclerotherapy, Post Veint Surgery, Post EVLT
- Edema

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Indication

- Lymphedema
- Edema
- Chronic venous disease (CVD)
- Chronic Venous Insufficiency (CVI)
- Venous Leg Ulcer
- Lipedema

- Basic liners non-compressive

Compression Level

21-46 mmHg

Includes

- Basic liners non-compressive
- Accutab® Device

Colours

Black

Liners for

Coolflex™ & Compreflex®

Basic liners

Non-compressive

Colour

Grey

Features

- Easy to apply knee-high liners
- *drirelease*® moisture wicking and odor control
- Flat seams for added comfort
- Silver ionized treatment for bacteriostatic properties
- Size: one size fits all

Included with

- Coolflex™ Standard Calf
- Compreflex® Standard Calf
- Also available separately

Transition liners

10-15 mmHg in foot and ankle

Colour

Black

Features

- Soft, flat seam knee-high liners
- *drirelease*® moisture wicking and odor control
- Mild compression in foot and ankle only
- Padded sole from toe to heel for added comfort
- Silver ionized treatment for bacteriostatic properties
- Sizes: sizing chart on page 26

Included with

- Compreflex® Transition Calf
- Also available separately

Complete liners

10-15 mmHg in foot, ankle and calf

Colour

Black

Features

- Comfortable knee-high liners
- *drirelease*® moisture wicking and odor control
- Mild compression in foot, and ankle calf
- Padded sole from toe to heel for added comfort
- Silver ionized treatment for bacteriostatic properties
- Sizes: sizing chart on page 26

Included with

- Compreflex® Complete Calf
- Also available separately

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Indication

- Lymphedema
- Edema
- Chronic venous disease (CVD)
- Chronic Venous Insufficiency (CVI)
- Venous Leg Ulcer
- Lipedema

Compression Level

21-46 mmHg

Includes

- cotton liners
- Accutab® Device
- Belt & Hip attachment (colour: black)

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Material composition

TRADITIONAL compression stocking:

70% polyamide, 30% elastodiene

Understocking: 62% polyamide, 26% elastane, 12% cotton

Donning and doffing

Applying the understocking

- One **understocking** with low pressure (15 – 20 mmHg)

+

- One **overstocking** with 23 – 32 mmHg of pressure

=

This **combination** exerts a total pressure ~ 45 mmHg² on the ankle.

Good to know

Contraindications for compression therapy

Relative

- acute weeping dermatoses
- intolerance to compression stocking material
- paraesthesia in the limb extremities
- severe peripheral neuropathy (e.g. diabetes mellitus)
- primary chronic polyarthritis

Absolute

- severe arterial occlusive disease
- congestive heart failure
- septic phlebitis
- phlegmasia cerulea dolens

Care

After wearing, the stockings should be washed by hand or at max. 40 °C (understockings: max. 60 °C) with a gentle wash cycle in the washing machine in a laundry net. It is important to use a mild detergent without fabric softener. The Sigvaris Washing Solution is suitable for washing by hand.

To preserve maximum durability of the product, please inform your customers that we do not recommend using a dryer/heater. The stockings must not be ironed or dry cleaned. The stockings should be stored dry at room temperature away from direct sunlight.

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Kontra Indikasi:

- Dermatitis
- Advance peripheral arterial occlusive disease

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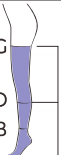
Black

TRAVENO product detail

- Intensive graduated support (15-18mmHg around the ankle region)
- Reduced swelling of the legs
- Breathable and easy-care
- Unisex

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Material : 71% Lateks, 28% Nilon, 1% Polyester

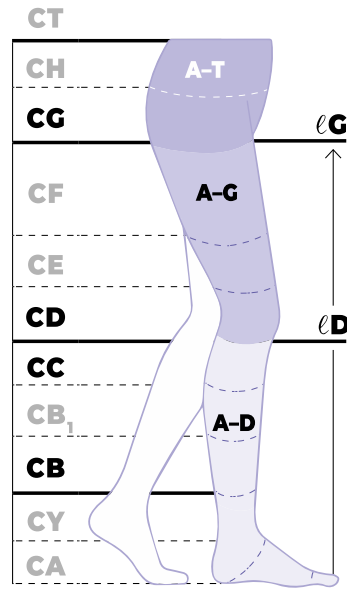
Color : Biru

A healthcare professional, wearing a white short-sleeved shirt and dark pants, is kneeling on a light-colored wooden floor. They are using a long, thin measuring tape to measure the lower leg of a patient who is standing. The patient is wearing dark-colored pants. The background is a light-colored brick wall. The entire image is overlaid with a semi-transparent blue filter. The text 'Form ukur' is prominently displayed in the center, with 'SIGVARIS COMPRESSION STOCKINGS' written below it in a smaller, all-caps font.

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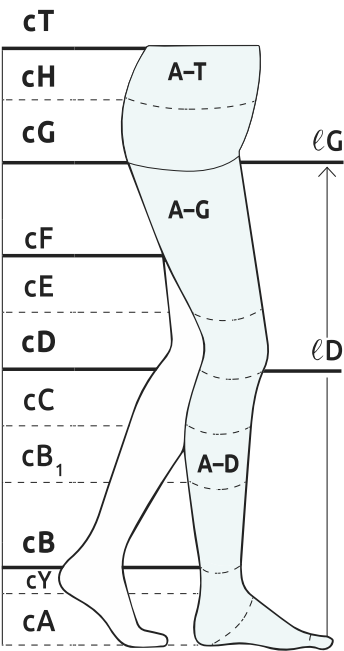
CG _____
CD _____
CC _____
CB _____

Size
Charts

	Ukuran Lingkar (cm)					
	S	S Plus	M	M Plus	L	L Plus
cT	up to 80	up to 80	up to 90	up to 90	up to 100	up to 100
cH	up to 110	up to 110	up to 120	up to 120	up to 130	up to 130
cG	42-52	50-60	50-60	58-68	58-68	66-76
cF	41-50	45-54	48-57	53-62	55-64	61-70
cE	32-37	36-41	35-40	40-45	38-43	44-49
cD	28-34	32-38	31-37	36-42	34-40	40-46
cC	31-36	35-40	34-39	39-44	37-42	43-48
cB ₁	24-29	24-29	29-32	29-32	32-36	32-36
cB	19-23	19-23	23-26	23-26	26-29	26-29
cY	28-32	28-32	32-35	32-35	35-38	35-38
cA	19-23	19-23	23-26	23-26	26-29	26-29

Panjang (cm)					
short	33 – 37	short	61 – 70	short	61 – 70
long	38 – 44	long	71 – 80	long	71 – 80
 A-D A-G A-T					

A-D : stocking sampai bawah lutut (calf)
A-G : stocking sampai paha (thigh)
A-T : stocking sampai pinggang (pantyhose)

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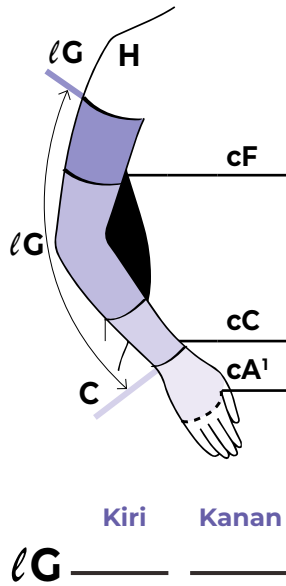
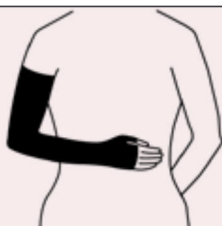
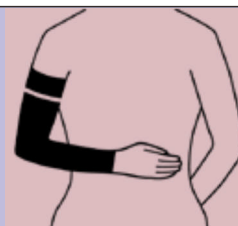
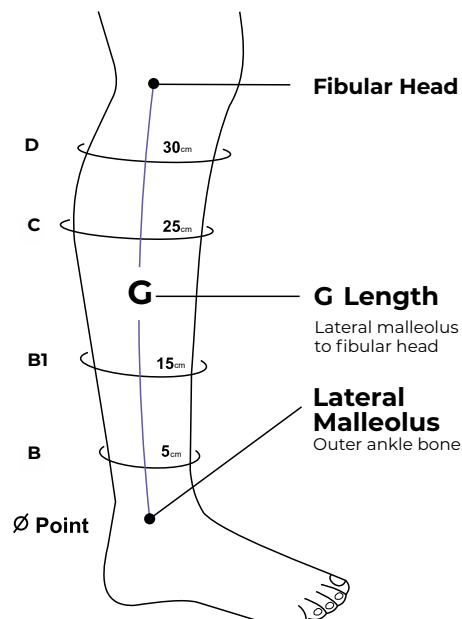
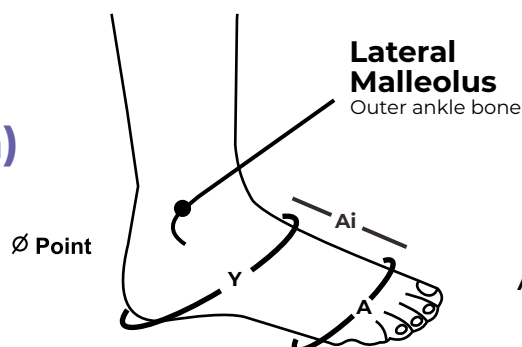
Ai _____

Measuring Instructions

Y: Measure circumference of ankle bend and heel
A: Measure circumference of across metatarsal heads
Ai: Measure lenght from heel to 1st metatarsal head

Ai: Measure medical length from heel to 1st metatarsal head

Compreboot Standard Foot Size Chart (I-K)

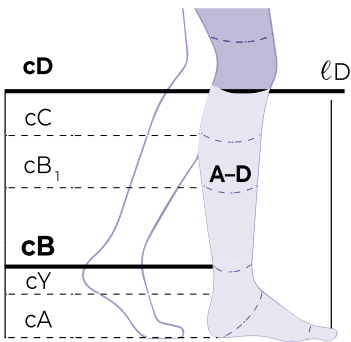
	Small		Medium / Large		X Large / XX Large	
	Regular	Long	Regular	Long	Regular	Long
Y	28-36cm	28-36cm	≤39cm	≤39cm	≤44cm	≤44cm
A	22-26cm	22-26cm	≤30cm	≤30cm	≤33cm	≤33cm
Ai	14-18cm	19-23cm	18-20cm	23-25cm	20-22cm	25-27cm

Nama Pasien : _____

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No. Telepon : _____

Nama Dokter : _____

Ukuran
Lingkar (cm)

Kiri Kanan

CD _____

CB _____

	Ukuran Lingkar (cm)									Length cm	
	XS	S	S Plus	M	M Plus	L	L Plus	XL	XL Plus		
cD	26-32	28-34	32-38	31-37	36-42	34-40	40-46	38-43	44-49	short	33 – 37
cC	28-31	31-36	35-40	34-39	39-44	37-42	43-48	40-45	47-52	long	38 – 44
cB ₁	21-24	24-29	24-29	29-32	29-32	32-36	32-36	35-39	35-39		
cB	17-19	19-23	19-23	23-26	23-26	26-29	26-29	29-32	29-32		
cY	25-28	28-32	28-32	32-35	32-35	35-38	35-38	38-41	38-41		
cA	17-19	19-23	19-23	23-26	23-26	26-29	26-29	29-32	29-32		

Material composition:
TRADITIONAL compression stocking:
70% polyamide, 30% elastodiene

Understocking:
62% polyamide, 26% elastane, 12% cotton

SCAN ME

Cara
Ukur

FORM UKUR

Inelastic Wrap - Knee Compreflex® Standard Knee

Nama Pasien : _____

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No. Telepon : _____

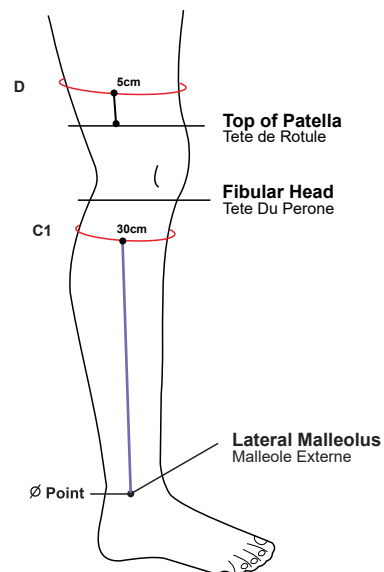
Nama Dokter : _____

Ukuran Lingkar (cm)

Kiri Kanan

D _____

C1 _____

Compreflex® Standard Knee (D-E)

	SMALL	MEDIUM	LARGE	X-LARGE
D	38-48 cm	46-56 cm	53-63 cm	58-73 cm
C1	29-39 cm	34-44 cm	39-49 cm	44-65 cm

SCAN ME

Cara Perawatan

FORM UKUR

Inelastic Wrap - Thigh Compreflex® Standard Thigh

Nama Pasien : _____

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No. Telepon : _____

Nama Dokter : _____

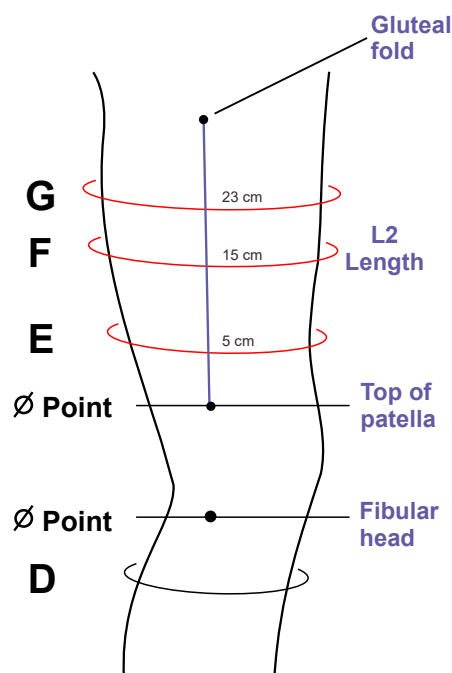
Ukuran Lingkar (cm)

Kiri Kanan

G _____

F _____

E _____

Compreflex® Standard Thigh (E-G)

	SMALL	MEDIUM	LARGE	X-LARGE
G	48-58 cm	56-66 cm	64-74 cm	74-84 cm
F	43-53 cm	51-61 cm	58-68 cm	68-78 cm
E	38-48 cm	46-56 cm	53-63 cm	63-73 cm
L1	REGULAR: 20-24.9 cm			

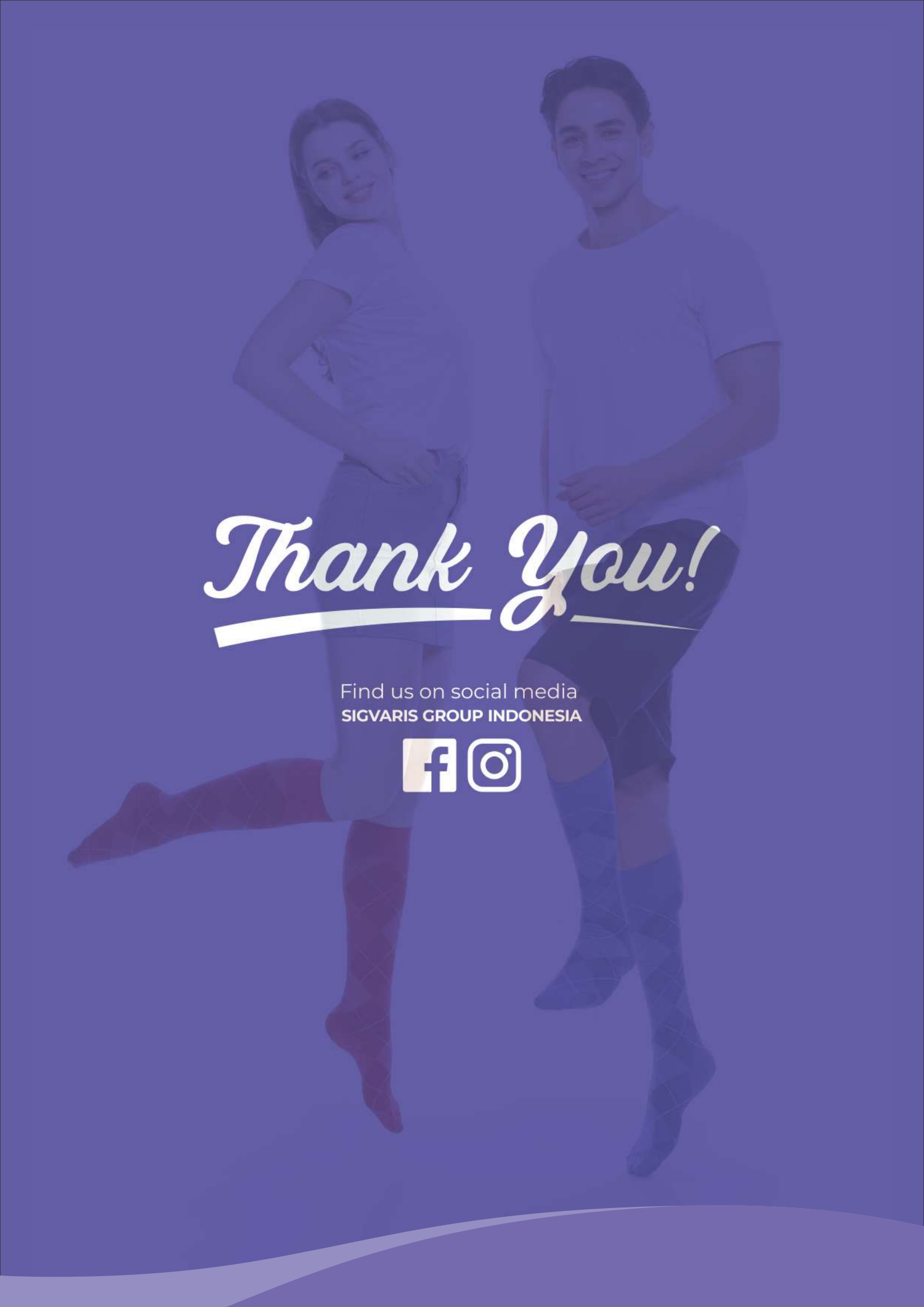
SCAN ME

Cara Pasang

Cara Perawatan

Thank You!

Find us on social media
SIGVARIS GROUP INDONESIA